



## UriSelect™ 4 Medium

Direct Identification Visibly Reliable




# UriSelect™ 4

Non selective chromogenic medium for the isolation, differentiation and enumeration of urinary tract infections pathogens within 18-24 hours:


- Isolation and identification of both Gram-positive and Gram-negative pathogens in a single plate
- Chromogenic mix allowing the detection of specific enzymes activities
- **Direct identification of *E. coli*, *Enterococcus* spp. and *Proteus mirabilis*** via revelation and visualization of enzymatic activity of bacteria
- Presumptive identification of KESC, Streptococci and Staphylococci (differentiation of *S. saprophyticus*)
- **Inhibition of *Proteus* spp. swarming** with an original formulation




## Assay Procedure

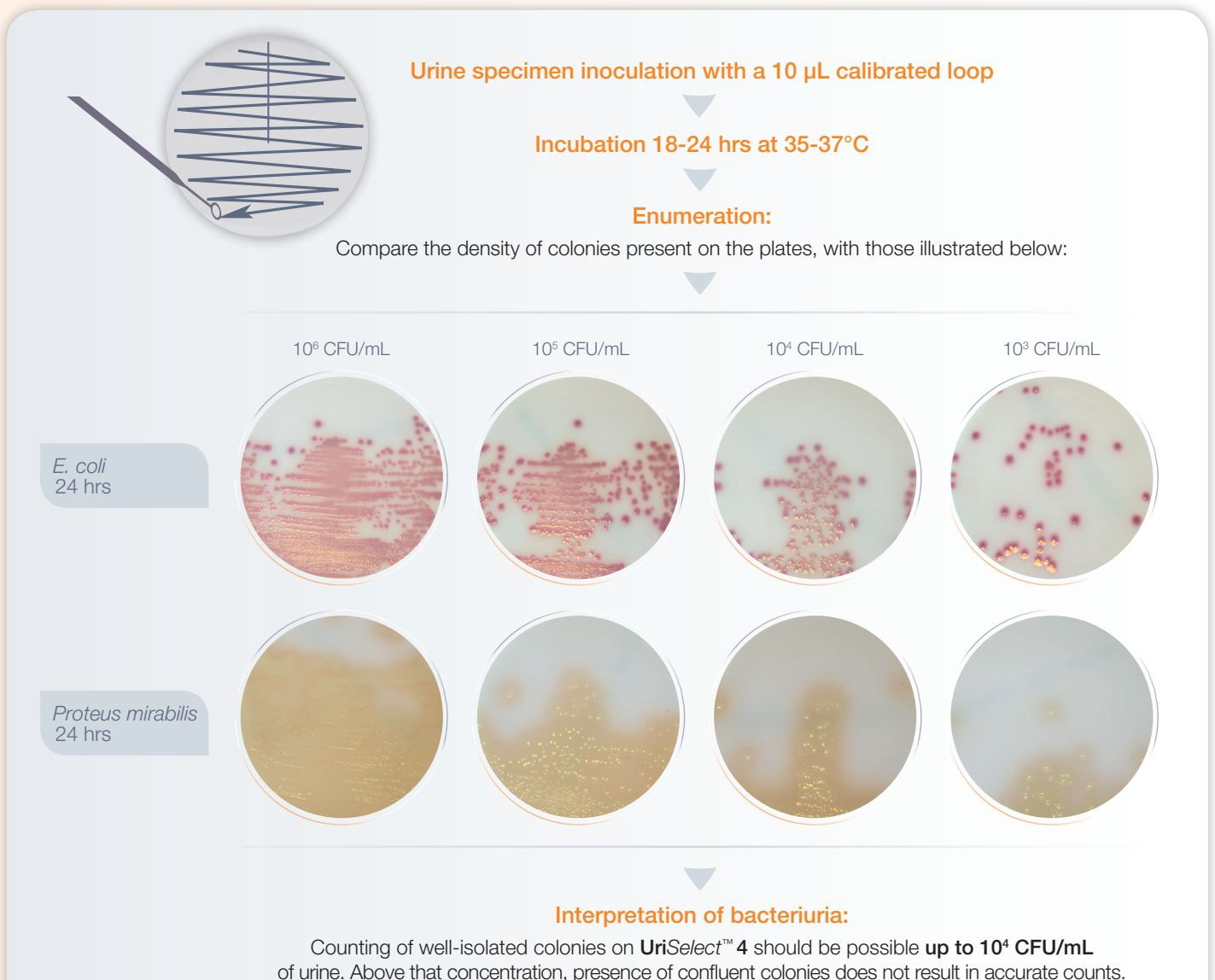





# Direct Identification of *E. coli*, *Proteus mirabilis* and *Enterococcus* spp.

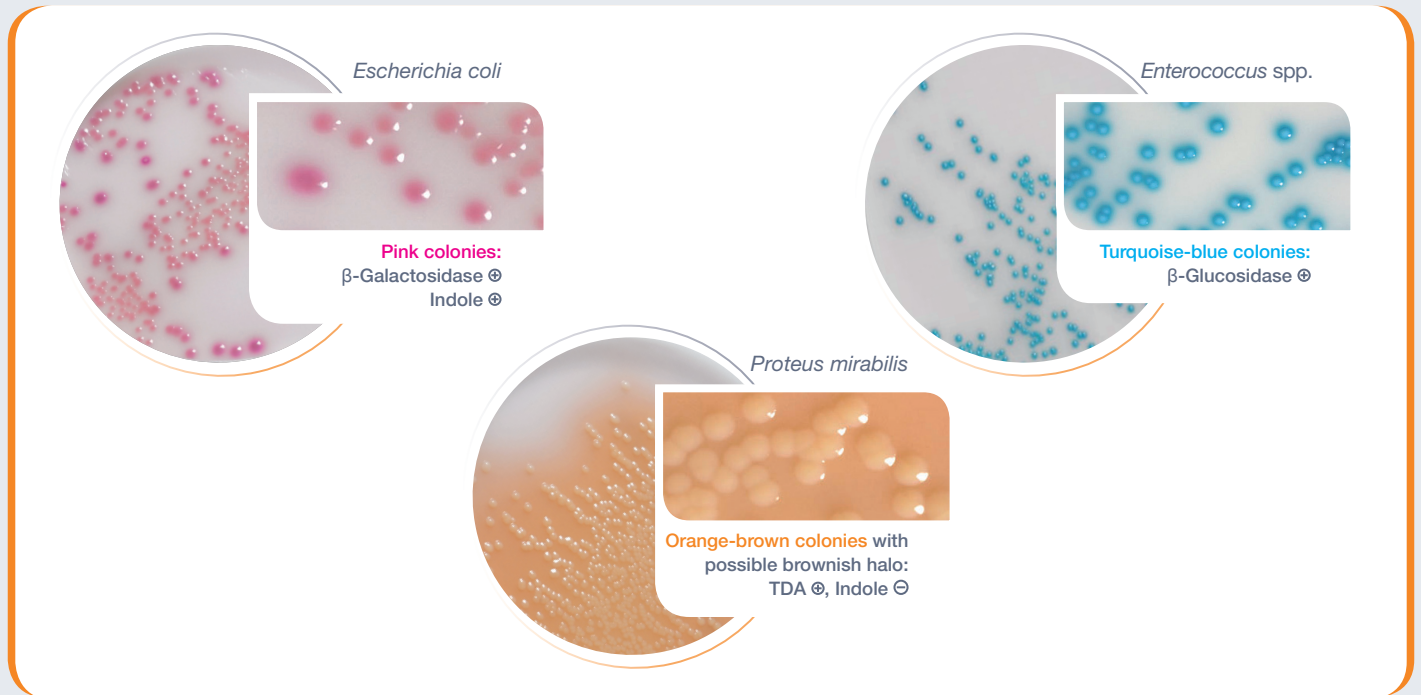


# Presumptive Identification of KESC

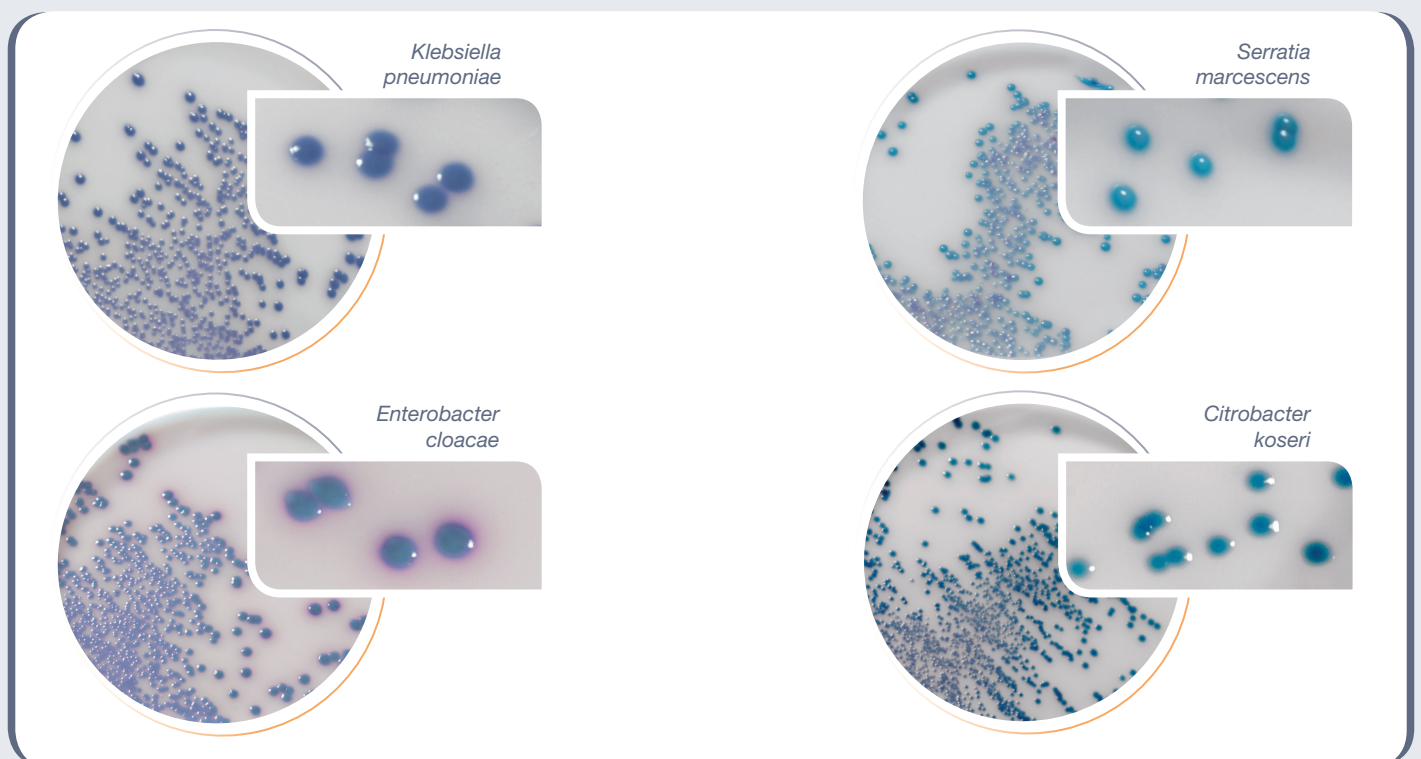

# Examples of Other Pathogens Growth

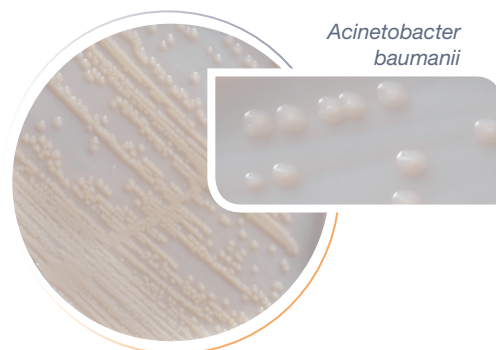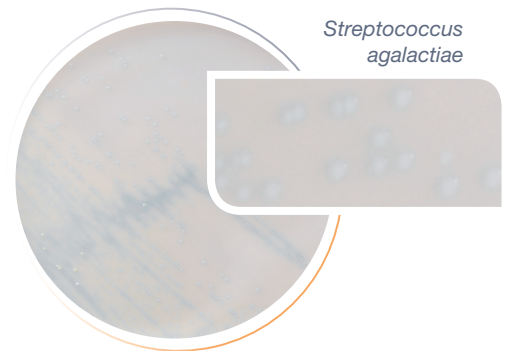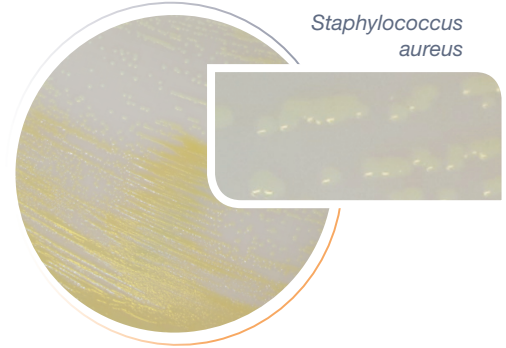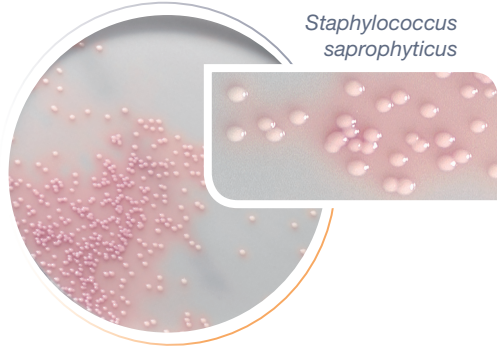


# Pathogens Identification

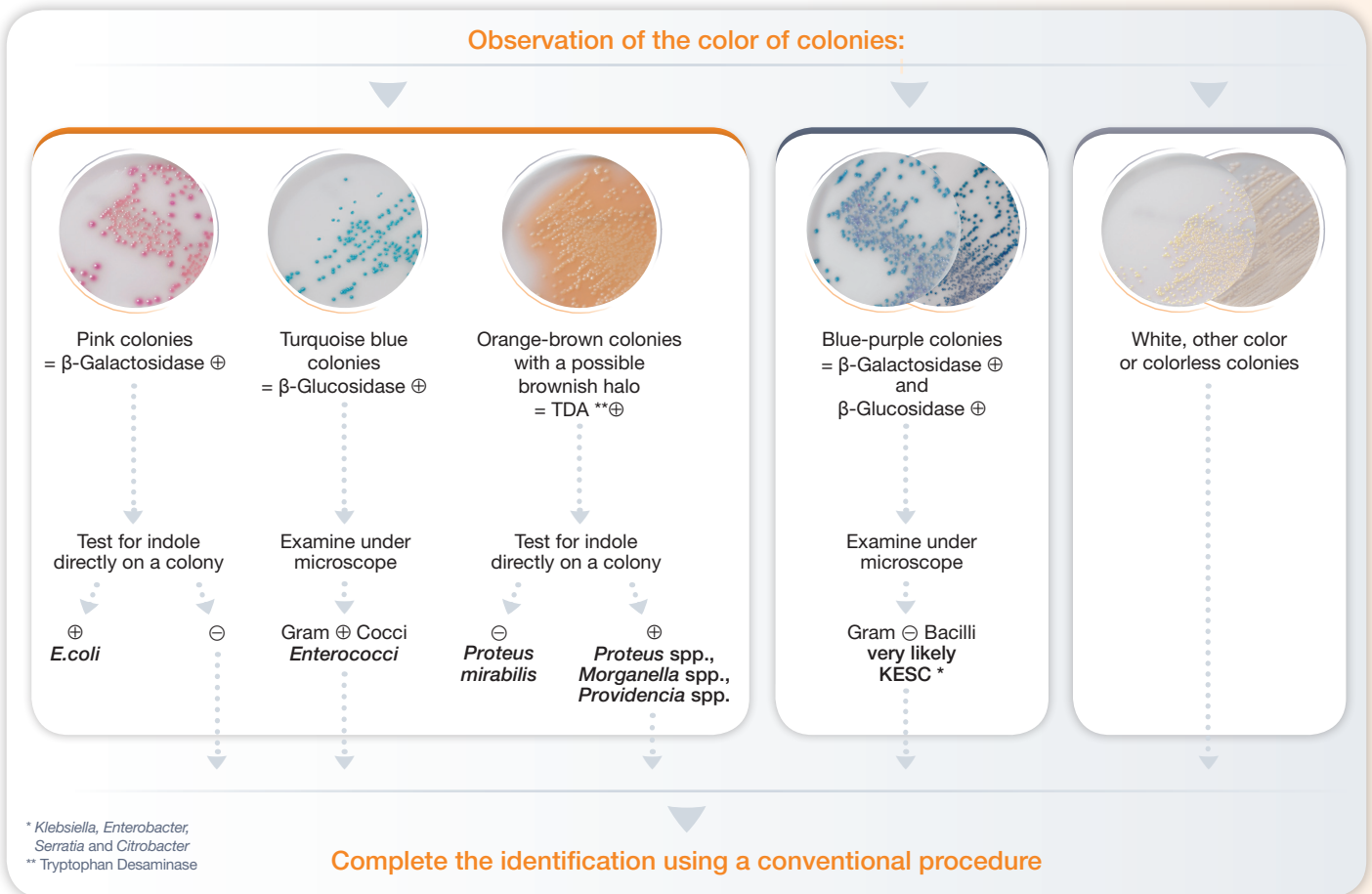

# Visualization and Identification of Mixed Cultures

Contrasting colors of colonies enhanced by a pearl-grey opaque background allow for an optimal visualization of polymicrobial mixtures.

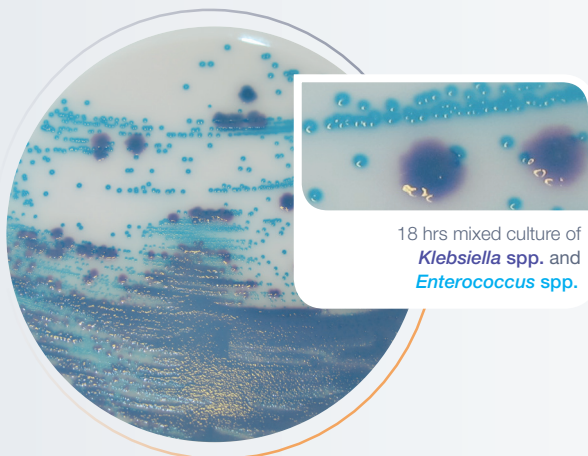

## Ordering Information

| Catalog #                       | Description                           |
|---------------------------------|---------------------------------------|
| <b>Urinary Tract Infections</b> |                                       |
| 63726                           | UriSelect™ 4, 20 plates x 90 mm       |
| 63727                           | UriSelect™ 4, 100 plates x 90 mm      |
| 64694                           | UriSelect™ 4 (dehydrated), 500 g      |
| 55313                           | Kovacs reagent, 2 vials x 15 ml       |
| 63713                           | Urea Indole medium, 50 ampules x 1 ml |
| 53834                           | O.X. (Oxidase), 2 x 50 disks          |
| 56350                           | Novobiocin (5 µg), 30 disks           |
| 56356                           | Pastorex™ Staph Plus, 50 tests        |
| 56353                           | Pastorex™ Staph Plus, 5 x 50 tests    |
| 61721                           | Pastorex™ Strep, 60 tests             |
| 56513                           | AuxaColor™ 2 kit, 20 tests            |

Visit [www.bio-rad.com/select-chromogenicmedia](http://www.bio-rad.com/select-chromogenicmedia) for more information.




| Catalog #  | Description             |
|--|-------------------------|
| <b>Rapid Detection of Resistance to Third Generation Cephalosporins for Enterobacteriaceae</b> |                         |
| 68250  | β LACTA™ Test, 50 tests |
| <b>Rapid Detection of Carbapenemase-Producing Enterobacteriaceae</b>                           |                         |
| 68260  | β CARBA Test, 25 tests  |
| <b>Rapid Identification of Carbapenemases Types</b>  |                         |
| K-15R1   | OXA-48 K-SeT*           |
| K-15R2   | KPC K-SeT*              |

\* Manufactured by  and distributed by Bio-Rad.

Ask for availability in your country.

### Other Chromogenic Media

|       |                                   |
|-------|-----------------------------------|
| 63757 | MRSASelect™ II, 20 plates x 90 mm |
| 63748 | SASelect™, 20 plates x 90 mm      |
| 63751 | VRESelect™, 20 plates x 90 mm     |
| 63750 | StrepBSelect™, 20 plates x 90 mm  |
| 63740 | CandiSelect™, 20 plates x 90 mm   |

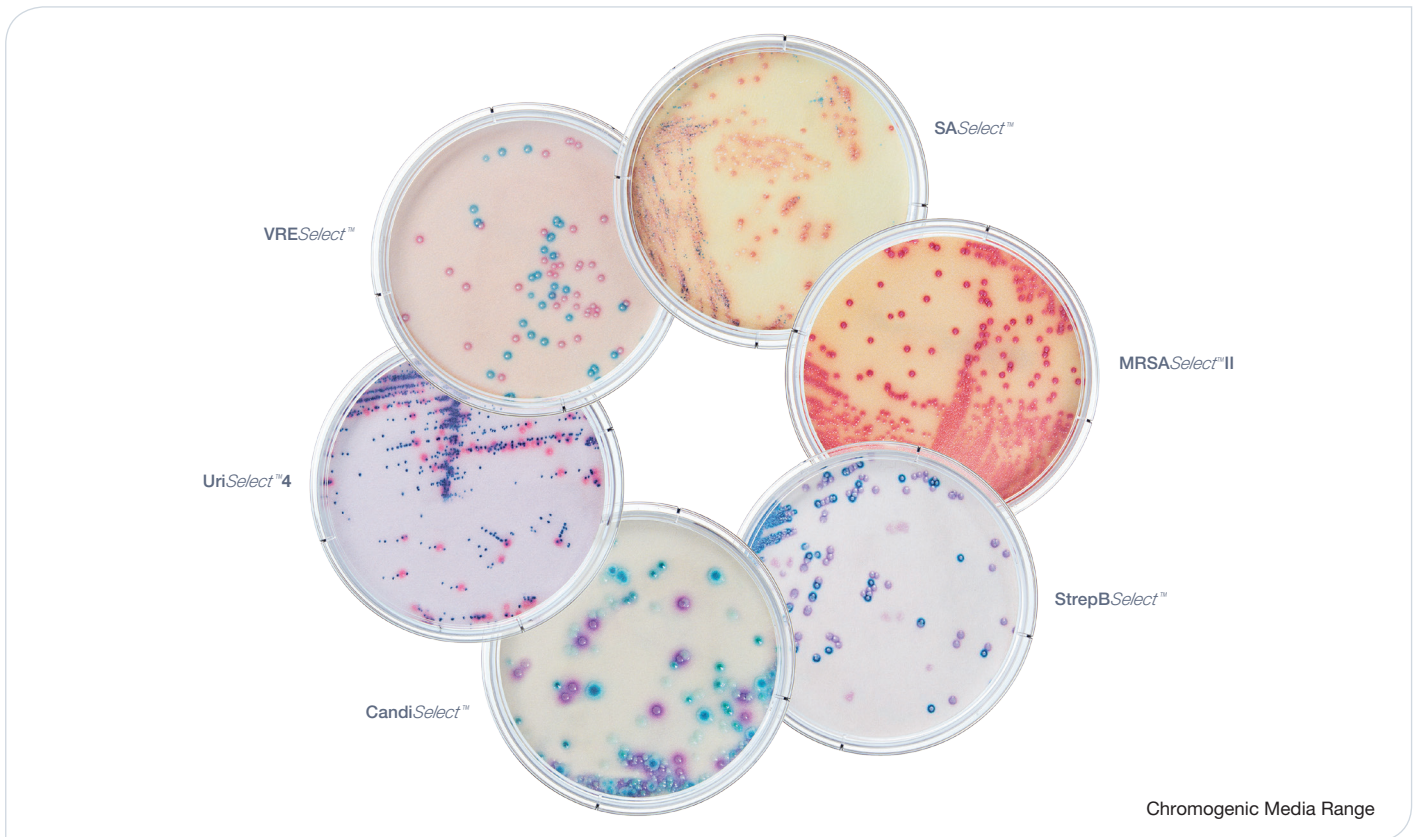

Chromogenic Media Range

**BIO-RAD**

**Bio-Rad  
Laboratories**

Clinical  
Diagnostics Group

Website [www.bio-rad.com/diagnostics](http://www.bio-rad.com/diagnostics) Australia 61-2-9914-2800 Austria 43-1-877-8901 Belgium +32 (3)710-53-00 Brazil +55 (31)3689-6600  
Canada 1-514-334-4372 China 86-21-61698500 Czech Republic 420-241-430-532 Denmark +45-4452-1000 Finland 358-9-804-22-00  
France 33-1-47-95-60-00 Germany +49 (0)89-318-840 Greece 30-210-7774396 Hong Kong 852-2789-3300 Hungary +36-1-459-6100  
India 1800-180-1224 Israel 972-3-9636050 Italy +39-02-216091 Japan 81-3-6361-7070 Korea 82-2-3473-4460 Mexico +52 (55)5488-7670  
The Netherlands +31-318-540666 New Zealand 64-9-415-2280 Norway +47-23-38-41-30 Poland 48-22-3319999 Portugal 351-21-472-7700  
Russia +7-495-721-1404 Singapore 65-6415-3170 South Africa 27-11-442-85-08 Spain 34-91-590-5200 Sweden 46-8-555-127-00  
Switzerland +41 (0)26-674-55-05/06 Taiwan 886-2-2578-7189 Thailand 662-651-8311 United Kingdom +44 (0)20-8328-2000

